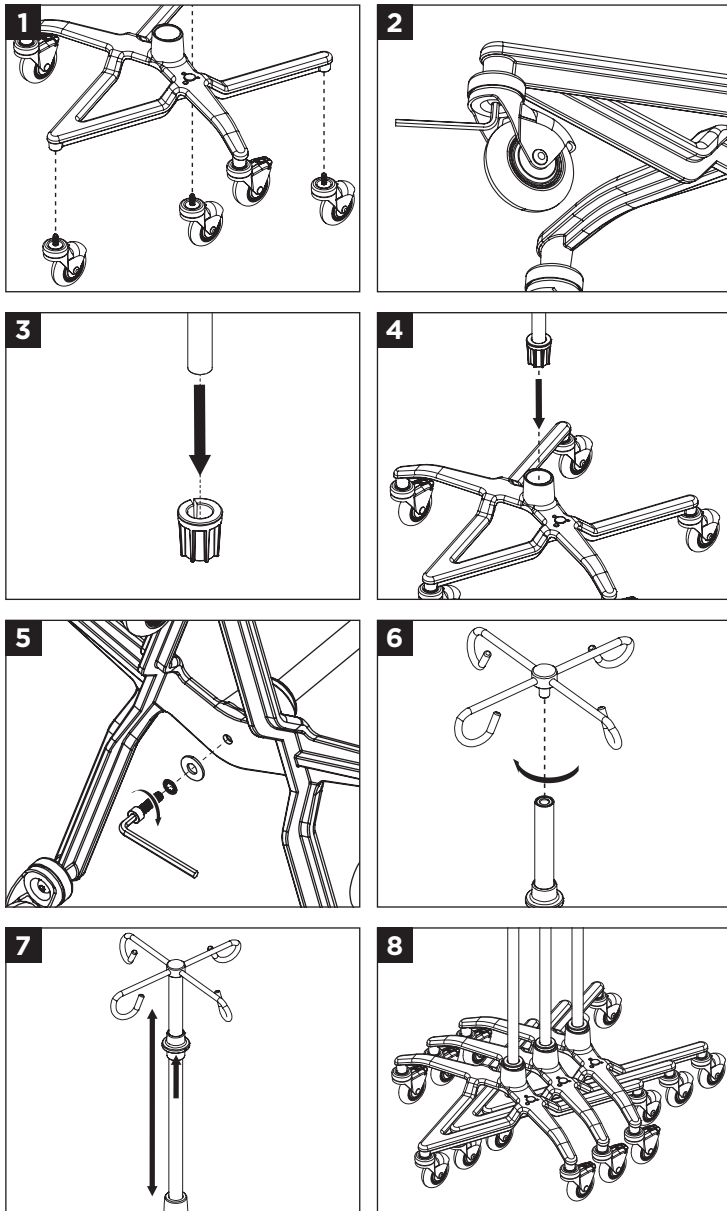
SmartStack

EN	Instructions for Use 2	NL	Gebruiksaanwijzingen 4
FR	Instructions d'utilisation . . 3	IT	Istruzioni per l'uso 4
DE	Bedienungsanleitung 3	ES	Instrucciones de uso 5

NOTE: The latest edition of this operating manual can be downloaded from our website at www.maxtec.com

CAUTION: Federal law restricts this device to be sold by or on the order of a medical professional.

EN

	Warning		Do Not
	Follow instructions for use		Lot code/Batch code
	Manufacturer		Catalog Number
	Federal law (USA) restricts this device to sale on or by the order of a physician		Authorized Representative in the European Community
	Medical device		MR Unsafe
	Date of Manufacture		Caution

WARNINGS

- ◆ This device is not MRI compatible.
- ◆ Never exceed 100 lbs on this device.
- ⊘ **DO NOT** attach more than 11 lbs (5kg) on one single hook. Distribute weight evenly.
- ⊘ **DO NOT** mount equipment that places the load more than 5 inches off center of the pole.
- ⊘ **DO NOT** mount more than 50 lbs on upper portion of this device.
- ◆ Use caution when rolling heavily loaded device, especially on uneven surfaces.
- ◆ This product should only be used under proper supervision from a healthcare professional.
- ⊘ **DO NOT** autoclave.
- ⊘ **DO NOT** stand on or ride this device.
- ◆ Brakes may not stop IV pole on all surfaces.
- ◆ Follow the order of assembly as directed.
- ◆ Use caution when adjusting pole height with heavy objects mounted above hands.
- ◆ If components are damaged or missing, contact your dealer immediately.
- ⊘ **DO NOT** use substitute parts unless provided by Maxtec or authorized representative.
- ◆ Failure to adhere to these warnings can result in severe injury.
- ◆ Maxtec assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- ◆ Use caution if pole is fully loaded. Product may tip.
- ◆ Use caution around stairwells.

CLEANING

- ◆ Clean often using a cloth moistened with 65% isopropyl alcohol/water solution or germicidal wipe.
- ◆ Clean the entire device before use.

MAINTENANCE

- ◆ Maxtec recommends inspecting this device for proper assembly and functions at least once a year.
- ◆ Inspect pole/base bolt, casters, and top hook and tighten if necessary.
- ◆ Ensure the pole height adjustment mechanism functions properly.
- ◆ If the device does not function properly or is missing components, remove it from service and contact Maxtec or authorized representative.

WARRANTY

- ◆ 4-year limited warranty on manufactured defects.

NOTE: Serious incident(s) that occur in relation to the device should be reported to Maxtec and the competent authority of the Member State in which the user and/or patient is established. Serious Incident(s) is defined as directly or indirectly led, might have led, or might lead to the death of a patient, user, or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat

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