



Equipped for success.

Heavy Duty I.V. Pole

REF USMEDIVP





Not made with Natural Rubber Latex.

General Warnings:

- **DO NOT** use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before use - otherwise serious bodily injury or product damage may occur.
- **DO NOT** exceed the weight capacity of 28lbs attached to the pole and 9lbs on each hook. (It is recommended that all added weight be evenly distributed amongst the top hooks to maintain stability.)
- All wheels must be in contact with the floor at **ALL TIMES**.
- **DO NOT** use the I.V. pole while walking backwards down gradients or climbing stairs, curbs or to go over obstacles. Serious risk of fall or injury may occur.
- Use care when moving I.V. pole with a patient. Avoid wet or slick surfaces, wires, cables, or tubes and any uneven surfaces that may cause I.V. pole to tip or fall.
- Only use accessories and spare parts authorized by US Med-Equip Industries.
- Caution should be taken if using multiple pumps. Pumps should be hung so that they counterbalance each other.
- **DO NOT** attach I.V. pole to other objects.
- **DO NOT** use in MRI areas.
- **DO NOT** use the I.V. pole for support or as a walking aid. Do not attempt to support your weight or use as a stand assist.
- **DO NOT** lift the pole by the top half of the pole.
- **DO NOT** use the IV pole for anything other than its intended use.
- **EXTREME CAUTION** should be taken when the IV pole is near the bed or any other furniture.

Parts Included:



- A. Base (1)
- B. I.V. Shaft (1)
- C. I.V. Hook Assembly (1)
- D. Flat Washer (1)
- E. Bolt (1)
- F. Spacer (1)
- G. Knob (1)
- H. Allen Wrenches (2)
- I. Lock Washer (1)

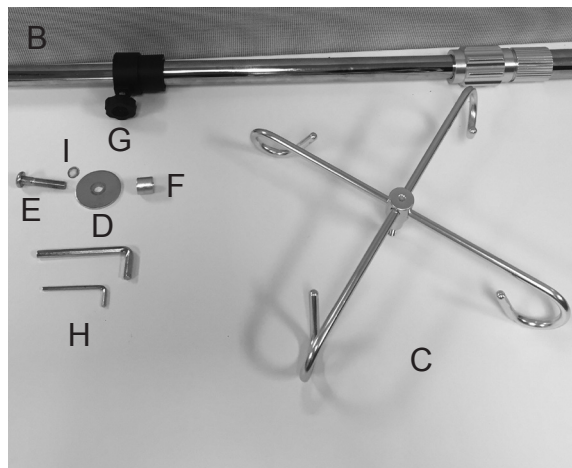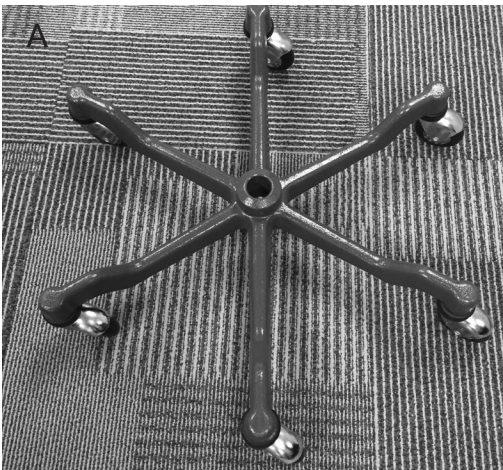


Note: Casters will already be assembled to the base.

Assembly Instructions:

1. Un-wrap all parts from packaging.
2. Assemble the bolt, lock washer, flat washer, and spacer. Connect the shaft to the top of the base by inserting the assembly into the underside of the base through the base. Tighten the bolt with the “Allen Wrench” provided.

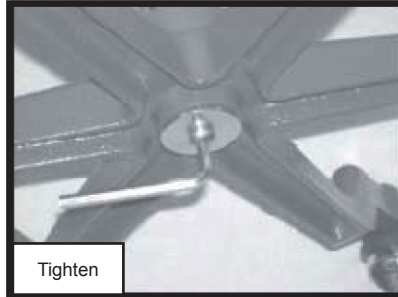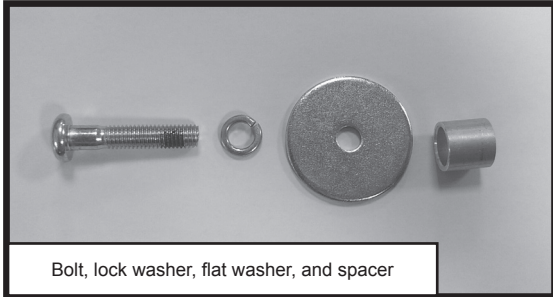

3. Install the I.V. Pole Height adjustment knob.
4. Install the I.V. Hook assembly by screwing the end into top of upper pole.

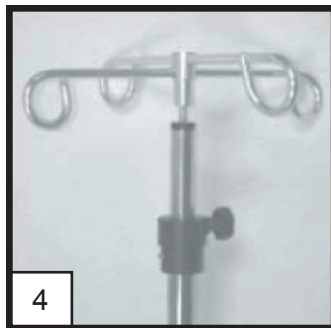

Maintenance:

1. Periodically check that the bolt is properly tightened in the bottom of the base and that the nuts on the casters are secure.
2. Cleaning of the I.V. Stand should be in accordance with the facility's maintenance policies.

Limited Warranty:

Your US Med-Equip product is warranted to be free of defects in materials and workmanship for a period of three (3) years from the date of purchase. Non-durable components such as hand grips and push-buttons are warranted for a period of one (1) year from the date of purchase. This express warranty extends only to the original purchaser. This warranty does not extend to damage or failure occurring as a result of owner misuse or negligence, normal wear and tear, or other causes unrelated to the manufacture and design of the product. This express warranty is US Med-Equip's sole and exclusive I.V.e warranty and your sole remedy in the event of default or failure. THIS IS THE SOLE AND EXCLUSIVE I.V.E WARRANTY FOR THIS PRODUCT. THERE ARE NO OTHER WARRANTIES, EXPRESS OR IMPLIED AND ANY SUCH WARRANTIES ARE DISCLAIMED. US Med-Equip is not responsible for any consequential or incidental damages or costs arising from the use of the warranted products, or any defect therein or failure thereof.



Made in China for: US Med-Equip, Inc.
7028 GESSNER RD HOUSTON,
TX 77040 713-983-8860
www.usmedequip.com