



# PROVIDING TRUE NEGATIVE PRESSURE WOUND THERAPY



# Providing Negative Pressure Wound Therapy, Mobility and Flexibility

Negative Pressure Wound Therapy is a preferred choice of care for treating acute and chronic wounds in hospitals, clinics, long-term care and home care settings.

Patients appreciate the ease-of-use, quiet operation and compact size of the Invia Liberty. The device can be carried in a discreet bag, facilitating mobility, and the docking station allows for easy charging.

Invia Liberty's innovative design delivers excellent functionality with its simple set-up and easy operation.

## Firmware 1.11



- Slower vacuum build-up time
- New air leak indicator
- User friendly mute alarm button
- Reduced time to detect and notify blockage in canister tubing
- Nightime mode for LED screen



- (a) Disposable canister 300 ml and 800 ml options with solidifier
- (b) One-touch canister release button
- c Easy to read LCD screen for intuitive operation and graphic capability
- d Double lumen tubing (PVC) with integrated protection filter and tube clamp
- (e) Covered, waterspray safe power adapter port
- (f) Covered, waterspray safe USB port for software updating and therapy history upload to a PC

### Indications for Use

Invia Liberty is indicated for use in:

- Diabetic/neuropathic ulcers
- Pressure ulcers
- Chronic wounds
- Acute wounds
- Dehisced wounds

# Contraindications

Do not use Invia Liberty for patients with any of the following conditions:

- Malignancy of the wound
- Untreated osteomyelitis
- Unmanged malnutrition
- Non-enteric fistula
- Unexplored fistula

Medela AG Lättichstrasse 4b

USA

Fax

6341 Baar / Switzerland

Phone +1 877 735 1626 +1 815 307 8942

www.medelasuction.com


suction@medela.com

www.medela.com Medela, Inc. 1101 Corporate Drive McHenry, IL 60050

- Necrotic tissue with eschar present
- Do not place Invia Wound Therapy Dressing over exposed blood vessels or organs.
- Do not place directly over anastomose or sutured blood vessels

Medela offers healthcare professionals 50 years of expertise in medical vacuum technology. Our Swiss development and manufacturing process yields mechanical precision to guarantee excellent performance.

### Technical data

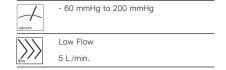

Medical Vacuum Technology for Healthcare Professionals

Please contact us or your local Medela representative for details.

Federal law restricts this device to sale or rental by or on the order of a physician.

Medela and Invia are registered trademarks of Medela Holding AG. Liberty is a trademark of Medela, Inc.

	> 12 Hours
kg 1	1 kg 2.2 lbs

Modes:	Constant and Intermittent
Tubing:	Double Lumen Tubing, Y-Connector
Canister:	300 and 800 ml
	H x W x D 95 x 170 x 150 mm 3.74 x 6.69 x 5.91 inch

2 year warranty

Local contact:

L	1
L	7
(	Ξ
(	Y
7	Г
(	Ξ
- (	7
1	`
1	`
(	כ
(	Y
7	_
(	_
-	-
ò	7
3	
(	7
	σ
-	σ q
-	ř
	4
-	2
	ñ