




Versatech 600/1100 Quick Reference Guide Including Scale & Bed Exit Alarm

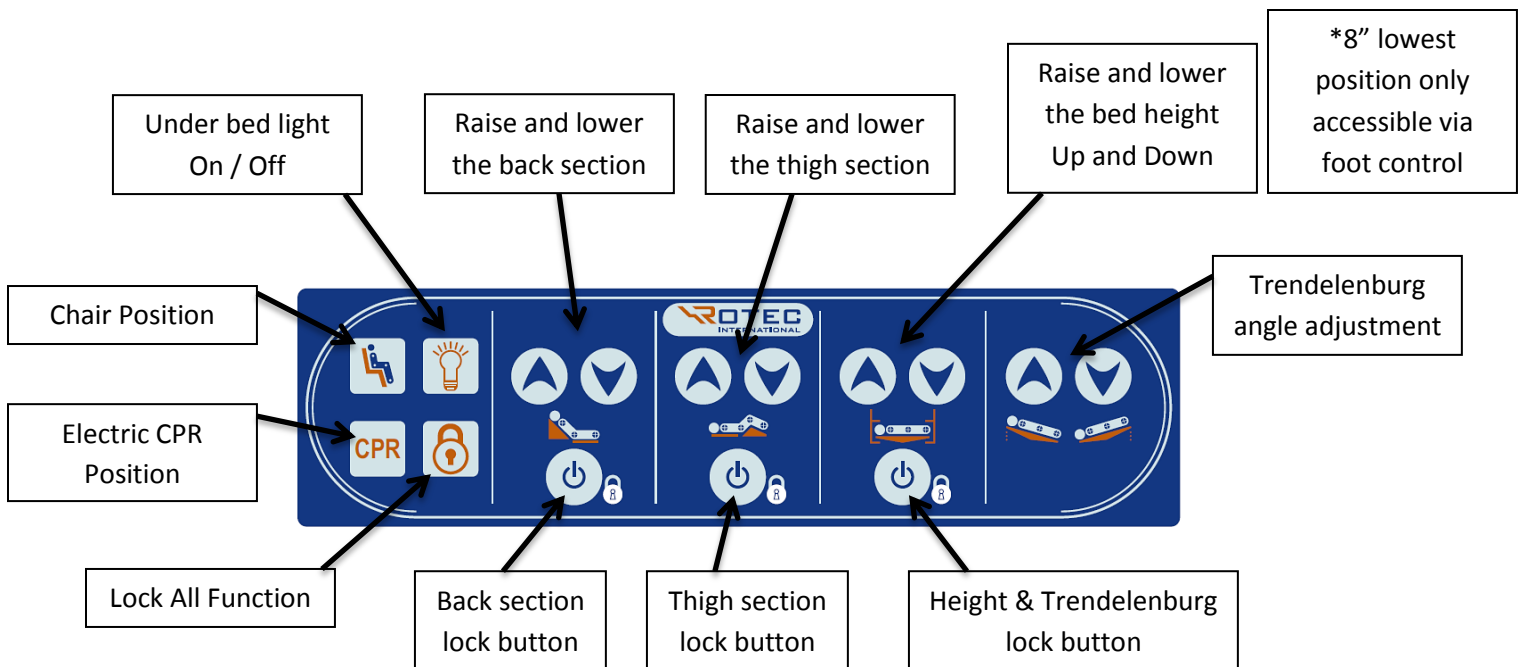


Versatech 600/1100 Quick Reference Guide

Including Scale & Bed Exit Alarm

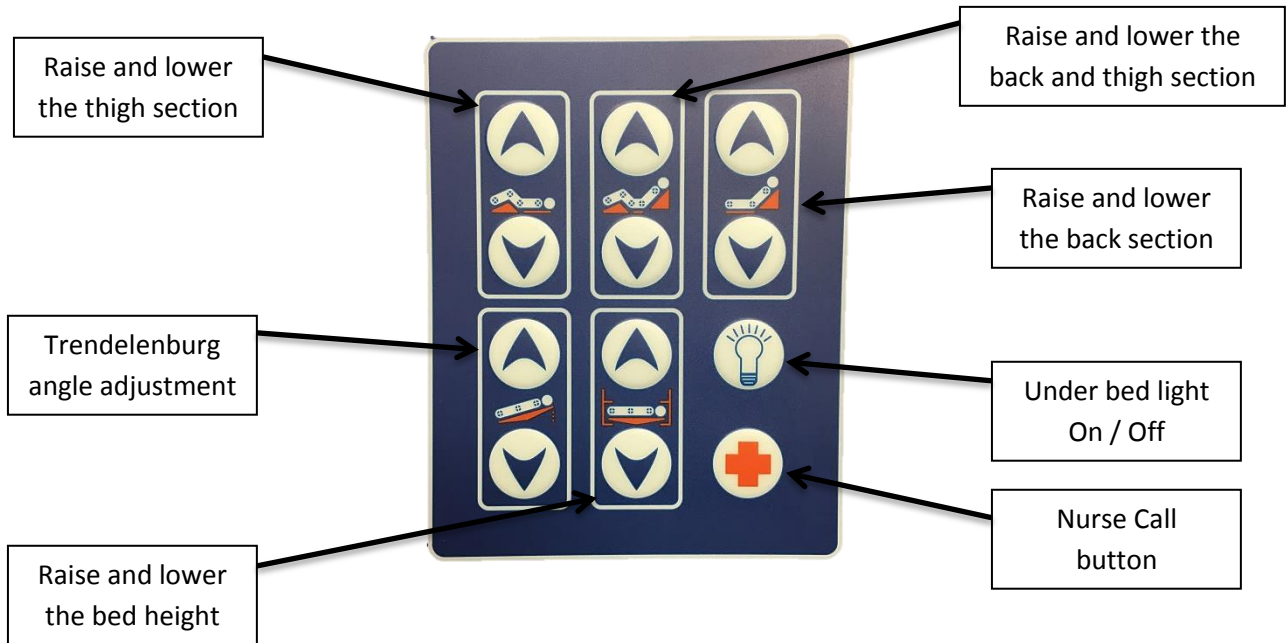



SETTING ZERO:

- It is recommended that the system be zeroed prior to each new patient.
- Ensure the patient is OUT of the bed and all equipment, linens, pillows, etc. are in place.
- Press and **hold** the left “ZERO”  button until “PLEASE WAIT, HANDS OFF” is displayed. Immediately let go and ensure nothing is in contact with the bed.
- The patient may now be placed in the bed.






WEIGHING:

- Press and **release** the “WEIGH”  button. Weight is displayed in both windows.





ADDING OR REMOVING WEIGHT:

- When the scale weight is stable, press and **hold** the “FREEZE”  button for 2 seconds. This will activate the “HOLD” mode.
- Add or remove any weight or accessories from the bed.
- The held weight will be displayed on the “WEIGHT” display.
- To save any changes press “FREEZE”  button.
- Press “FREEZE”  button to save or press “WEIGH”  button to cancel changes.


SETTING UNITS:

- Press “MENU”  button and use “UP”  button to scroll to “UNITS” on the display. Use “DOWN”  button to select “Lb” or “Kg”.
- Press “FREEZE”  button to save or press “WEIGH”  button to cancel changes.

SETTING THE BED EXIT ALARM:

- Press “MENU”  button to enter bed exit calibration.
- Press “DOWN ARROW”  button to select the desired alarm sensitivity (0 to 45 % in 5% increments. This is percentage of patient weight)
- Press “FREEZE”  button to save or press “WEIGH”  button to cancel changes

ARMING OR DISARMING ALARM:

- Press “EXIT ALARM”  button to enable or disable alarm. (There is a slight delay when the button is pressed prior to activation)
 - When Bed Exit Alarm is enabled, “A” is shown in the “Weight Change” Display in area indicated.



For detailed operating instructions, please refer to the user manual

If the bed &/or scale is malfunctioning, please contact your local Freedom Medical Branch or our Bed Service Team at:

484.879.4272 / bedservice@freedommedical.com

www.freedommedical.com



Corporate Beds
610 Jeffers Circle
Exton, PA 19341
p: 484.879.4272 f: 484.879.4396
e-mail: bedservice@freedommedical.com